
INSTRUCTION LEAFLET

F077 V1 02/2017

(PLEASE READ CAREFULLY BEFORE USE)

Use one thickness for prosthetic work or two for crown and bridge work. Soften in warm water (45°C). Remove and absorb excess moisture from the surface with a tissue. Knead into a mass if desired. Use to take accurate bite registrations and chill in the mouth with water or compressed air. Molten wax may burn.

Caution: Disinfect the wax before sending it to a dental laboratory.

Made in England by Associated Dental Products Ltd,
Purton, Swindon, SN5 4HT, UK.

t : (+44) 01793 770256 f : (+44) 01793 772256

sales@kemdent.co.uk www.kemdent.co.uk

